


# SOLUSCOPE CLN

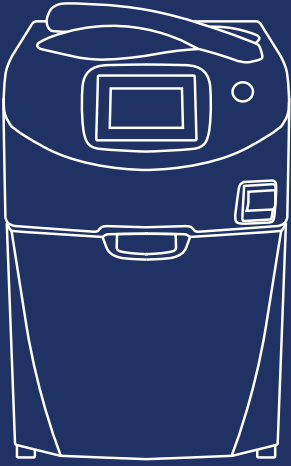

## PH NEUTRAL DETERGENT

PROVEN EFFICACY  
AGAINST BIOFILM  
AND PROTEINS

COMPLIANT WITH  
EN ISO 15883-4

COMPATIBLE WITH  
A LARGE RANGE  
OF FLEXIBLE  
ENDOSCOPES

LOCK KEY SYSTEM  
AND CANNULA  
INTEGRATED TO THE  
CONTAINER

## INDICATIONS

- Detergent for flexible endoscopes.
- To be used exclusively with *Soluscope Serie 4 PA*.

## COMPOSITION

- Association of surfactants with controlled foam
- Sequestering agent
- Excipients

## CHARACTERISTICS

- Pale yellow clear liquid
- Density at 20°C: 1.02mg/ml
- pH pure product at 20°C: about 6.9\*
- pH diluted at 1%: about 7.8\*
- Conductivity at 0.5%: 315µS/cm

\* pH value measurements under operating conditions always by means of a pH meter with temperature compensation

## USER MANUAL


- Put the container in place.
- Screw the white cap connected to your *Soluscope Serie 4 PA* onto the container (refer to the Washer-Disinfector user manual).
- Make sure that the vent hole is not blocked.
- Automated dilution: 1% (10mL/L), 35°C, 90 sec.
- Shelf life after opening: 2 months while connected to the Washer-Disinfector (write the opening date on the label in the location provided for this purpose).

## MEDICAL DEVICE USE SAFELY

- Please read label and product information.
- Medical Device.

## STORAGE

Dangerous – respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

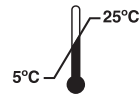

## PACKAGING

- 2 containers of 5L



D0737E-EN



**SOLUSCOPE S.A.S**  
(An Ecolab company)  
100, rue du Fauge  
Z.I. Les Paluds - 13400 Aubagne  
France



[www.ecolab.com](http://www.ecolab.com)



[www.soluscope.com](http://www.soluscope.com)